



## CleanCert water sample record

In order to ensure your waterline cleaning regime is effective and water quality is being maintained, it is necessary to perform dipslides **every 1-3 months** on ALL your dental unit waterlines immediately after you do a biofilm removal (shock) treatment. Any dipslide used needs to measure cfu/ml (TVC). Product code BFCDS10.

**Step 1** = Open tap and immediately remove the dipslide paddle from the outer tube and insert the 'tongue' into the waterline being tested. Avoid touching it with your finger, as this will invalidate the result.


**Step 2** = Shake off any excess water and replace back into test tube.

**Step 3** = Store out of direct sunlight for 7 days, at room temperature.

**Step 4** = If the diagram does not resemble image 1 below, email [technical@cleancert-hygiene.co.uk](mailto:technical@cleancert-hygiene.co.uk) We will then send you back a short questionnaire, which should quickly identify where the issue lies and what action needs to be taken.

**Step 5** = When logging the results on the monthly record sheet, you must tick which image your result matches.

**Step 6** = Keep the results for internal procedures like staff training, CQC inspection and management reviews. There is no need to send your CleanCert water sample record sheet back to CleanCert Hygiene. For further info, email [technical@cleancert-hygiene.co.uk](mailto:technical@cleancert-hygiene.co.uk)

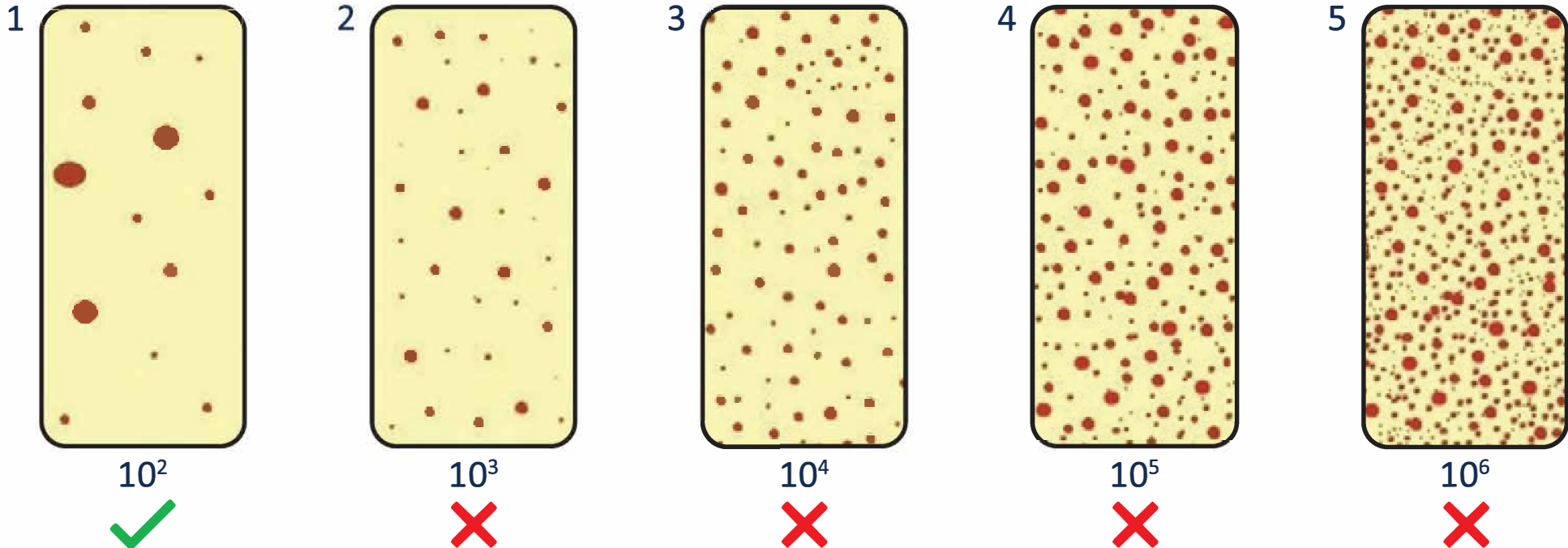



# CleanCert water sample record

202_	January					February					March					April					May					June									
	Tick to show which diagram your sample resembles					Tick to show which diagram your sample resembles					Tick to show which diagram your sample resembles					Tick to show which diagram your sample resembles					Tick to show which diagram your sample resembles					Tick to show which diagram your sample resembles									
Location	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5
<i>Example: S1</i>																																			
<b>Date &amp; Sign</b>	<i>Eg. 10.01.2021 - M .Smith</i>																																		
<b>Action Taken</b>	<i>Eg. Emailed technical@cleancert-hygiene.co.uk 01.01.21</i>																																		

